



Secura 1 Mental Health Bed

Instruction Manual | Product Code: 7-SEC1



Introduction:

This instruction manual contains information on the instructions for use and explanations of the signs and symbols used. These instructions for use contain instructions for operating the medical care bed.

These instructions for use may contain inaccuracies or printing errors. The information provided here will be updated periodically

and changes due to product maintenance will be included in later editions. Changes or improvements are possible at any time without prior notice. If you have any questions, please contact our customer service team.

The instructions for use must be read and used by every person who operates the medical bed.

Purpose

The intended purpose of the Secura 1 Mental Health Bed is to be used for patients in hospital and healthcare facilities. These beds are electrically powered to allow the bed to be placed in various positions suitable to the patient. It is a medical device intended for human medical use only.

The intended user of the Secura 1 Mental Health Bed is any

individual who is being cared for in a mental health facility.

The bed should only be used by individuals trained or experienced in proper handling. Users must be instructed on its correct use and familiarize themselves with these instructions. Common users include nursing staff, physicians, therapists, family members, technicians, and cleaning staff.

Symbols Explained




Manufacturer



For Indoor Use (House)

Product Serial Number Plate

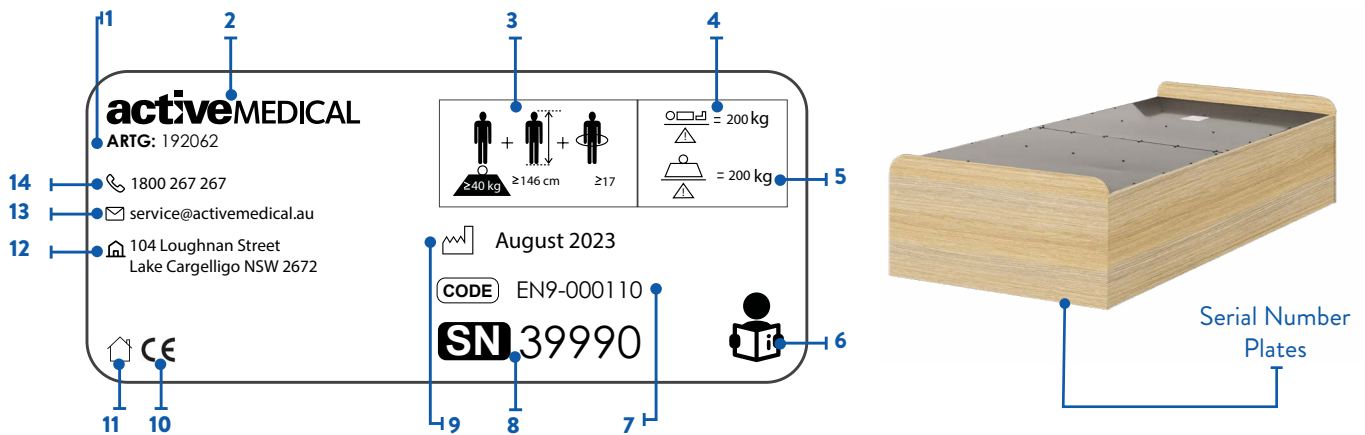

Serial Number Plate Explained

1	ARTG Number	8	Product Serial Number
2	Company Logo	9	Manufacture Date
3	Adult Identification (minimum weight, minimum height, minimum BMI)	10	CE Marking
4	Maximum Occupant Weight	11	For Indoor Use Only
5	Maximum Safe Working Load	12	Manufacturer Address
6	Read Instruction Manual - Follow Instructions For Use	13	Company Service Email Address
7	Product Code	14	Company Phone Number

MODELS AVAILABLE

Secura 1 Mental Health Bed - Single Width
 Secura 1 Mental Health Bed - King Single Width

SPECIFICATIONS

Size	Single	King Single
Overall Dimensions	920 x 2070 mm	1090 x 2070 mm
Sleeping Surface	900 x 2000 mm	1070 x 2000 mm
Mattress Base Height	450 mm	450 mm
SWL	200 kg	200 kg
TGA Number	ARTG 192062	
Manufacturer 	Active Medical, 104 Loughnan Street, Lake Cargelligo, NSW 2672	

**** LOAD CAPACITY HAS BEEN TESTED IN ACCORDANCE WITH AS/NZS 3200.2.38:2007 ****

 For Indoor Use (House)

MAINTENANCE

EVERY 6 MONTHS:
 ✓ Check there is no visible damage

Annually:

- Professional service of product to be carried out annually & documented in 'Record of Performed Service Procedures and Regular Service Maintenance' in the back of this instruction manual.

CLEANING

- Cleaning can be done often – using domestic detergents.
- Do not use acids, alkaline or any other chemical solvents.

Warranty

Conditions

Your Secura 1 Mental Health Bed is guaranteed for 10 years on the frame, from the date of purchase

Professional service of product to be carried out annually and documented in the 'Record of Performed Service Procedures

and Regular Service Maintenance.

This warranty applies only on the parts manufactured or approved by Secura.

Exclusions

This warranty does not extend to those items that may need replacement due to normal wear and tear, or to damage caused by misuse or accident, for which Secura or its distributors cannot be held responsible.

If your bed should need attention due to failure as defined under the warranty above, please contact the distributor from whom you purchased your bed.

Please fill in and retain for your own records:

- DATE PURCHASED:**
- MODEL:**
- SERIAL NUMBER/S:**
- DISTRIBUTOR:**
- PHONE:**
- EMAIL:**

- E & O.E
- Specifications subject to change without notification



SECURA

1800 267 267 | www.secura.au