

StaebeL Crash Mat User Manual

Product Code: 9824



Thank you for choosing Staebel Crash Mat.

We aim to ensure you are satisfied with your Crash Mat in terms of both performance and durability. We therefore kindly ask you to spend a few minutes reading through these user instructions to ensure you can take the best possible care of your Crash Mat.

If you have any questions that are not answered in these user instructions, please contact us at Staebel. You will find our contact details on the bottom page of these user instructions.

USING THE CRASH MAT

Simple precautions can help create a safer environment. The Staebel Crash Mat helps to minimise injuries from a fall. With its bevelled edges, the Crash Mat easily accommodates to wheelchairs, walkers, and table trays.

Specifications		
Product Code	9823	9822
Overall Width	600 mm	910 mm
Folded Width	600 mm	910 mm
Overall Length	1760 mm	1760 mm
Folded Length	880 mm	880 mm
Overall Thickness	40 mm	40 mm
Folded Thickness	80 mm	80 mm

INSTRUCTIONS FOR USE

1. Unfold the Floor Crash Mat.
2. Make sure the “TOP” label is facing up.
3. Position the Crash Mat lengthwise, with the straight edge flush to the side of the resident’s bed.

Warnings

- Inform resident and staff to take care when stepping on or off of the Crash Mat to avoid tripping.
- Follow your facility’s procedures for cleaning spills in patient care areas.
- DO NOT place Crash Mat on or around damp or wet areas. Clean spills on, around or under Floor Crash Mat. Liquids under Floor Crash Mat may cause a slip and fall accident.
- When used with a Low Bed, be sure to leave enough room between the bed and the Crash Mat. If the Crash Mat is too close when the handrail is lowered, it could tear the Crash Mat cover.

NOTE: Due to the random possibility of falls, the Staebel Crash Mat makes no guarantee, express or implied, that the user is protected from hip trauma.

CLEANING & STORING

Care & Maintenance

General Cleaning: Regularly clean your Staebel Crash Mat with a mild solution of phenolic based cleaner and water. Rinse thoroughly with fresh water and wipe dry.

Spot Cleaning: For difficult spills and stains, prepare a strong solution of phenolic based commercial cleaner. Using a soft bristle brush and solution, clean the affected area. Rinse thoroughly with fresh water and wipe dry.

Disinfection: It is common to disinfect surfaces in a healthcare environment. Using a water/bleach solution of 10:1, clean the entire mattress. The bleach must not have a sodium hypochlorite level in excess of 5%. Disinfectants applied at full concentration will decrease the life of Staebel Crash Mat. Rinse thoroughly with fresh water and wipe dry.

Notes:

- Never exceed the cleaning manufacturers concentration recommendations.
- Do not use abrasive cloths or cleaners.
- Do not use aggressive cleaners or chemicals.
- Staebel Crash Mat are not autoclavable.
- When in doubt, contact your local Staebel Distributor for further advice.

Storage & Handling

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

DISPOSAL

Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

