

Staebel Haven Rise Lift Chair User Manual

Product Code: 5572



Thank you for choosing Staebel Haven Rise Lift Chair.

We aim to ensure you are satisfied with your chair in terms of both performance and durability. We therefore kindly ask you to spend a few minutes reading through these user instructions to ensure you can take the best possible care of your chair.

If you have any questions that are not answered in these user instructions, please contact us at Staebel. You will find our contact details on the bottom page of these user instructions.

Before to do a cycle (to go up and especially to go down) – Make sure that nothing is caught when activating the mechanism (people, animals, object...)

The Staebel Haven Rise must be stored away in a dry and temperate place, while respecting the upright position storage of the cardboard box.

SAFETY INFORMATION

Do not operate the equipment without having:

1. Be initially trained on a practical test by the reseller or the installer
2. Having read and understood the user's manual.

Ensure a minimum distance (of about 60 cm) behind the back of the seat to prevent it from coming into contact with the wall when set in the 'Relax position'.

Please systematically make sure that nothing can be blocked or hidden under the mechanism before each using's cycle. The mechanism of the lift chair may cause personal injury, be very careful with young child or domestic animals.

Ensure that the seat is on a flat horizontal surface in order to perfectly stabilise the seat on the ground.

Do not place the seat close to a significant heat source.

Plug in the power supply cable to the electricity network in compliance with the current standards and matching the 220V-power-supply-unitvoltage.

In case of threat of thunderstorms, it is recommended not to use the seat.

The lift chair must not be moved when it use.

We suggest first adjusting the seat while it is unoccupied to become familiar with its operation. The user must ensure that children do not use the seat.

The leg rest is not an additional seat.

Before moving the seat around, unplug the power cable.

If the quilt is packed, please detach it from the chair and gently tap it to locate the source of the swelling.

Do exclusively use original spare parts and accessories distributed by Staebel, thus guaranteeing the product's security and maintaining its conformity.

It is forbidden to modify the seat, in which case the warranty will no longer be valid.

The miss using of the seat will also cancel the guarantee terms.

Do not expose the product to the sun, as it can result in a partial discolouration of the coatings. Do not remove the 2 rear wheels.

Attention!

In this user manual you will find warnings. These are clearly identified by the symbols and colour of the text boxes, see below for examples.

Indicates a potentially hazardous situation which may lead to death or serious injuries if it is not avoided

Appropriate Precautions

The use of the seat is not intended for disoriented people, and for users in need, the help and vigilance of a third party are imperative.

Weight Limit

The lift chair cannot be used by people weighting over 160 kg (model Staebel Haven Lift Chair 1 or 2 motors) 130 kg (XS) and 220 kg (XXL).


ASSEMBLY

Remove the chair from the packaging.

Begin to put the back rest on the base before to adjust the arm rests. Recline the mechanism of your chair behind with the remote control.


1. Lift the seat-padding in order to well see the bracket.
2. Set the brackets of the backrest (picture 1A) face to face to the brackets of the base (picture 1B). 2 seat-depths are available: 45 cm or 55 cm.
3. It's possible that the bracket of the base (B) may be offset slightly, you just have to 'pull' it in order to place it in front of the bracket of your choice (45 or 50 cm) from the backrest (A).
4. Slide the bracket (B) on the bracket (A) until the 'clac'.
5. Now you can put the arm rest on the correct place.
6. Insert the metal rod of the arm rest into the hole on the armrest board. Do it again for the other side.

Picture 1

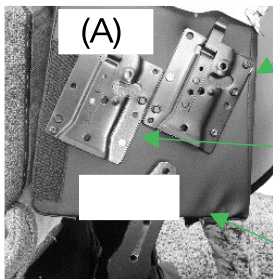

- Bracket (seat-depth 45 cm)
- Bracket
-
- Bracket

Picture 2

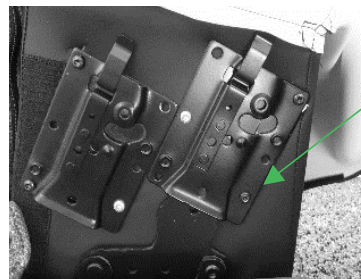

The backrest is fixed

1. Plug the transformer to the electric actuator's cable.
2. Plug the electric cable of the jack to the power transformer on the other end. Please check that all electrical connexion are in the correct place.
3. Plug in the power supply cable to an electricity network in compliance with the current standards and matching the 220V-power-supply-unit voltage.
4. If you want to put off the back rest, recline the mechanism on your chair on the back, then pull slightly the latch of the bracket outside, in order to put off the backrest.

Improper handling will not be covered under the guarantee.

ACCESSORY ASSEMBLY (ADDITIONAL ITEMS)


Magazine Holder Assembly

- Open the armrest (picture 1 bis).
- Put the magazine holder in the armrest board by taking notice to not trouble the closing of the armrest (picture 2 bis).

Picture 1 bis



Picture 2 bis

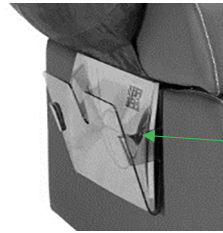

Put your magazine holder in the middle of the armrest board in order to not close the notch in front of the armrest board


Tray Assembly

- Put the lift chair Staebel Haven Rise in rising up position (picture 3)
- Fit the metal support of the tray into the opening predicted for (picture 4)
- Put the tray on the support.

Picture 3



Picture 4

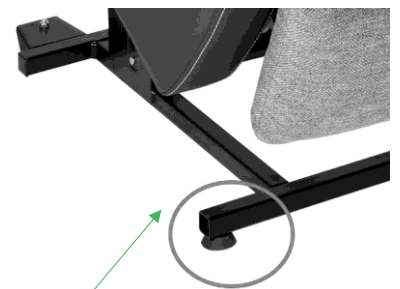

Opening for installing the tray's support

If you want to change the side of the tray, put off the black plugs which obstructs the opening and install your tray's support

USING THE REMOTE CONTROL

The remote control cannot drop, thanks to its placement under the armrest cushion.



Picture 5

For the Staebel Haven Rise single motor, XS et XXL (picture 5)

Remote control 2 buttons

The remote control 2 buttons allows to manage by one self its reclining

Top button : Reset in seated position with lowering of the leg rest, then after move to the rising up position

Bottom Button : Backward tilt with rising of the leg rest.



Picture 6

For the Staebel Haven Rise 2 motors (picture 6)

Remote control 4 buttons

Green button : Back reclining.

White button : back rising in straight position

Blue button : Rising of the leg rest. Return in sitting position

Yellow button : Downgrading of the leg rest. Switching in lifting position

Check that anything would be blocked or hidden under the mechanism before each cycle of using. The mechanism may cause serious injury to people, be very alert with young child or domestic animals.

Warning

When connecting your chair on power supply, DO NOT PRESS the remote control buttons. However, a power failure or another, would put your remote-control in 'safety mode' (locked), please to reset the remote-control by doing a full cycle of reclining of the chair: from the front raised to the complete back reclined or unplug your chair during 60 seconds.

Composition of the chair

Your Haven Rise consists of the following components:

1. Seatback
2. Seatback quilt washable/replaceable
3. Seat base
4. Seat base/leg rest quilt washable/replaceable
5. Leg rest
6. Two armrests retractable/removable
7. Comfort cushion reducer seat 6
8. Two armrests boards
9. 2-buttons remote-control reversible left/right - Stop Start function




Accessories

It is possible to have the following optional accessories :

- Tray reversible left /right (picture 7)
- Magazine holder reversible left /right (picture 8)
- Hoist friendly kit (picture 9)
- Mobility kit (picture 10) fitted with a flange + 1 transport stick




Picture 7

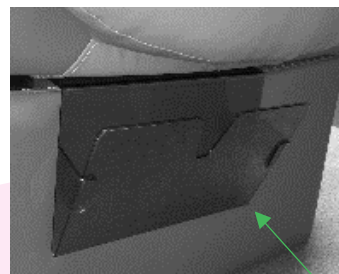

Picture 8



Picture 9

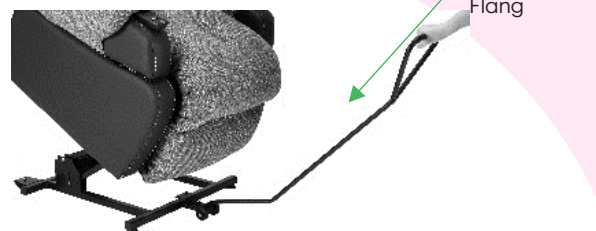

Picture 10

Transport Stick
Flang

Changing the quilt modules to be washed or replaced

Seat Back Quilt

- Remove the Velcro strap from the flap of the cover behind the seat back
- Zip down the left and the right side on the quilt

Seat Base/ Leg Rest Quilt

- Remove the Velcro straps at the bottom, on the right and on the left of the seat base.

All modules can be machine-washed at 30°C

We advise you to put the modules in a washing net or in a pillowcase, to preserve their original aspect

Changing the seat back:

It is possible to modify the seat base depth by replacing the back rest with another one

- Drawn on the « CLIP » and remove the seat back by sliding it away from its base along the 2 metallic rails
- Assembly the new seat back to the base by sliding it down along the 2 metallic rails until a perfect engagement of both parts, until the « CLIP »

Specifications:

Maximum User Weight	160 kg (1 or 2 motors) / 130 kg (XS model) / 220 kg (XXL model).
Coatings	In compliance with the norms anti fire 1021-1 et 1021-2: PVC/PU: Chocolate, Taupe grey, Licorice, Pistachio, Raspberry, Test report n°367160734 and n° 367160735 Polyester (velvet): Brown and Raw Test report n°18.0433.01 and n°18.02686.01 Mottled fabric: Brown and Grey Test report n° 18.03363.01
Quilt Padding	Silicone hollow fiber

Cleaning Protocol for Coatings

- All detergent, solvent and stain remover not listed on the NF collectivity WILL BE REALLY PROSCRIBED
- Ball-point pen's ink spot, require an immediate cleaning in order to prevent from the migration into the coated fabric and becomes also definitive.

GUARANTEES & COMPLIANCES

Mechanism and actuator: 5 years

Chair : 2 years

The chairs StaebeL are manufactured in total accordance with the regulations as mentioned, included the last amendment and the law which arrange these amendments:

- Low-voltage electrical equipment 2014/35/UE
- Electromagnetic compatibility (CEM) 2014/30/UE
- Restriction of the use of certain hazardous substances in electrical and electronic equipment RoHS Directive 2011 / 65 / EU

The chairs StaebeL complies with all electric standards as following:

- EN 60335-1/A11:2014
- EN 55014-1/A2 :2011
- EN 55014-2/A2 :2008
- EN 61000-3-2 :2014
- EN 61000-3-3 :2013
- EN 62233:2008
- EN 50581 :2012

ATTENTION



- Do not let children play with your lift chair
- Always let the lift chair and the leg rest closed after used
- Make sure hands and foot away from the mechanism
- Only the end user must to operate the lift chair
- Put the lift chair not less than 80 cm from the wall

Please keep this notice