

# Staeibel Folding Shower Chair User Manual

Product Code: 1350



## **Congratulations on your purchase of an Staebel quality product.**

Before using your product, please read all instructions carefully. The warranty will not apply if the product has been used incorrectly. This product has been tested to a safe working load of 100kg. Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

## **INTRODUCTION**

### **Intended Use**

For use in a bathroom or shower, to provide a stable seated surface for clients with reduced mobility who are unable to stand during showering. It can be folded for storage when not in use.

### **Safety**

- The maximum user weight must not be more than 100 kg.
- Ensure all four legs are level and in contact with the shower floor.
- Test lightly with hands before sitting. Sit centrally on the shower stool.

## **ASSEMBLY/INSTALLATION**

### **Unpacking**

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

### **Unfolding/Folding the Frame**

Open the frame so that the arms are pointing forwards, 90 degrees to the back rest. Raise the seat frame and fit into the locations on the arm sections. Ensure they are correctly located. Test with your hands to ensure the chair is stable and secure. To fold; raise the seat frame from the locations and lower. Turn one arm section into the seat frame and the other rotate behind the back rest.

### **Height Adjustment**

Adjust to the correct height by removing the pin clips, sliding the leg extensions up/down the frame tube to the appropriate height and replace the pin clip. Ensure all four leg extensions are secured to the same adjustment hole to ensure even height.

## **CLEANING & MAINTENANCE**

### **Maintenance**

Inspect at regular intervals for wear and damage.

### **Cleaning Instructions**

Clean regularly, wiping down with mild detergent /disinfectant solution. Allow product to fully dry prior to use/storage.

## PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

## WARRANTY

Staebel guarantee this product for a period of **12 months**, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only. Any guarantee issued by Staebel does not affect your statutory rights.