

Staebel Strato3 Padded Commode Seat User Manual

Product Code: 19781 & 19790



Thank you for choosing Staebel Strato3 Commode Seat

Your Commode Seat belongs to a family of high-quality products intended to be used in a variety of care environments, such as senior/assisted living, special care facilities, medical facilities, hospitals, nursing homes, rehabilitation units and other types of healthcare settings. Do not hesitate to contact us in case of queries about the operation or maintenance of your Staebel equipment.

Thoroughly read and understand this user manual before using your Commode Seat. Information from this user manual is essential for proper operation and maintenance of the equipment.

SEAT INSTALLATION (For Juvo Commodes ONLY)



1. Locate 4 x Seat Klipps and Screws (Juvo registered design) and #2 Screw Driver.



2. Locate fixing point lugs to suit your shower commode chair. Fix Klipps to the bottom of the seat.



3. Push down on seal firmly until clipped in position. Pull up on inside of seat to remove.

CLEANING INSTRUCTIONS

Vinyl Fabric

General Cleaning:

It is recommended that all Staebel vinyls and P.U's be cleaned weekly/fortnightly using a sponge/cloth with warm soapy water. Dry with a clean cloth as this will remove dust and general soiling that can stain the product over time.

Spot Clean:

For spot cleaning, Staebel recommends a 50:50 solution of methylated spirits and water. Follow with a solution of warm soapy water to remove any residue. Most stains will lift off easily.

Bleach Cleaning:

All Staebel vinyl and P.U's are suitable for hospital applications and as such can be cleaned using bleach based cleaning products. A 10% household bleach (Sodium Hypochlorite) with 90% water solution (500ml bleach with 5L water) can be used to disinfect/sanitise our products.

Ink Stains:

Ink stains, ballpoint pens & permanent marker; rub the marked area with undiluted isopropyl alcohol (rubbing alcohol) then immediately rinse the cleaned area with freshwater promptly after applying the alcohol. NOTE: it's not guaranteed that ink stains can fully removed. To achieve the best results, it is recommended to clean the stain immediately.

Dye transfer:

Dye transfer from blue jeans and other items of clothing can stain the surface of light coloured vinyls. It is important to treat the stain promptly using the Spot Cleaning Method. If the dye penetrates the surface of the vinyl, it will be harder to completely remove.

Dartex Fabric

General Directives:

Attention must be given to the characteristics of any other materials that may be combined with Dartex in the final product (e.g. dimensional stability, colour fastness, washing instructions).

- Some surface wrinkling may result from cleaning procedures. This has no adverse effect on Dartex properties.
- Abrasive cleaning agents should not be used.
- Additionally, due to the range of cleaning equipment and conditions being used, customers should ascertain that any Dartex fabric does perform as expected, in any given situation.
- If customers have their own particular cleaning methods that must be used and that are not covered in this article, they should consult Carr Australia Pty Ltd for further direction.

Washing and Disinfection:

Dartex needs to be treated with care when cleaning the surface. Many industrial cleaning agents are extremely harsh on the coating and can cause it to break down. Please refer to the list of cleaning products (available from the Carrgroup website) that have been tested and approved for use on Dartex. If your particular cleaning solution does not appear please contact us as we can test Dartex with any submitted cleaner. We will respond with our findings and recommendations.

- To avoid premature degradation of the surface coating never use an abrasive cloth, sponge or other similar device on the surface. Carefully follow manufacturer instructions for recommended exposure time of cleaner on surface. Wipe off cleaning product residue with a clean, soft cloth after cleaning and allow the surface to dry completely before use.
- The normal cleaning instructions are to wipe the surface clean with a sponge or cloth with warm soapy water.
- Dartex is able to be washed in warm soapy water at up to 95°C.
- Do not use solvent on Dartex.
- Dartex is able to be autoclaved at a temperature of 134°C for 3 minutes. However, care must be taken to first clean the surface with pure water and then to make sure mattresses are not stacked on top of each other without any air gap separation. Spacers should be placed between mattresses when a number of them are being autoclaved at the same time.
- Dartex is dry-cleanable with all dry cleaning fluids except trichloroethylene.

Ironing - Do not iron Dartex.

Storage - Store in a cool dry area and avoid excessive pressure and contact with non-colourfast materials.

WARRANTY

The warranty period of the product is **12 months**. The warrant will be deemed void if it is established that our product has been:

1. Intentionally damaged due to vandalism.
2. Not been properly installed or upholstered in accordance to industry standards.
3. Not been properly maintained or regularly cleaned as per cleaning instructions.
4. Altered using chemical surface or base treatments.

5. Exposed to permanent inks & dyes.
6. Post treated but not limited to flame or adhesive lamination using incompatible chemical adhesives, foams and methods.
7. Cleaned using inappropriate cleaning agents and methods.
8. Exposed to external contaminants or unsuitable cleaners.
9. This warranty only applies to the original purchaser from Staebel and shall not extend to any subsequent purchaser or other third party.

In order to make claim under this performance warranty within the warranty period. Staebel requires that a written claim, photographic evidence and a sample of the material in question be provided within 30 days of discovery. Staebel makes no express, implied or statutory warranty of merchantability or fitness for any other purpose other than for the purpose Staebel products are intended.