

Staebel Mobile Overbed Table User Manual

Product Code: 4215



Congratulations on your purchase of an Staebel quality product.

DO NOT use this product without first completely reading and understanding these instructions. If you are unable to understand warnings, cautions or instructions, contact a healthcare professional, dealer, or technical personnel before attempting to use this equipment.

Specifications	
Product Code	4215
Top Width	450 mm
Top Length	750 mm
Min Height	725 mm
Max Height	1050 mm
SWL	20 kg



OPERATING THE OVERBED TABLE

Adjusting Table Height:

1. Squeeze the height adjustment lever up toward the table top.
2. Adjust the table top to the desired height.
3. Lightly press down on the table top until it locks into place. An audible 'click' will be heard when mechanism is secure.



CLEANING & MAINTENANCE

- Clean the device regularly and keep it clean.
- Use a damp cloth and mild detergent, and dry with a clean cloth.
- Lighter dirt and dust can be removed with a wet cloth. Dry with a clean cloth.
- Do not use harsh, caustic or abrasive cleaners as this will void the warranty.
- Always rinse off soap residue after use.

WARRANTY

Staebel warrants this product to be free from defects in materials and workmanship for **one year** for the original purchase. If within such warranty period this product shall be proven to be defective, it shall be repaired or replaced, at Staebel's discretion. Product repairs shall not extend this warranty.

This warranty shall not apply to problems arising from normal wear and tear or failure to adhere to the enclosed instructions, subject to negligence, accident, improper operation, maintenance or storage, modification without manufacturer's written consent.

For warranty service, please contact Staebel.