

Staebel 360HR Horizontal Bed Rails User Manual

Product Code: 7-360HR



Thank you for choosing the Staebel 360HR Horizontal Bed Rails. Please carefully read these instructions before using the rail. This instruction manual contains information on the instructions for use on operating this accessory.

These instructions for use may contain inaccuracies or printing errors. The information provided here will be updated periodically and changes due to product maintenance will be included in later editions. Changes or improvements are possible at any time without prior notice. If you have any questions, please contact our customer service team.

The instructions for use must be read and used by every person who operates this accessory.

PURPOSE

The intended purpose is to be used for individuals requiring support for mobilisation, repositioning and transfer requirements. Users must be instructed on its correct use and familiarise themselves with these instructions. Common users include nursing staff, physicians, therapists, family members, technicians, and cleaning staff.

PRE-OPERATION CHECK

With the rails in position, check the following:

- Make sure this product doesn't interfere with moving bed parts.
- When fitting safety sides to this bed please beware of the possible crushing points.
- The clamps can create crushing points on the rails.

Important:

Warning: The word Warning refers to a hazard to unsafe practice that may cause severe injury or death to you or another person. Accident risk Warning: Do not use or allow use where the intended user has;

- Cognitive impairment
- History of recurrent falls
- Uses medication that would compromise falls
- Limited mobility comprises safe use

Designed to be fitted to Staebel P5500 Bed. Failure to comply may result in warranty being void.

Specifications

Overall Length - Upright / Folded	1400 / 1837 mm
Overall Height - Upright / Folded	400 / 40 mm
Material Clamping Size	31.6 x 50.8 mm
TGA Number	ARTG 217347

FITTING THE RAILS



- 1.** Remove clamp plate completely.
Slide rails over bed frame



- 2.** Once rails are placed over bed frame, make sure there is approximately a 60mm gap between the rails and the headboard.
Make sure this product doesn't interfere with moving parts.



- 3.** Once in position, tighten the clamps, using a 17mm spanner.
NOTE: Check regularly to ensure tight.



- 4.** Test rails before use.
To fold rails down, pull down on white knob.

MAINTENANCE

- Rails must be inspected regularly for any wear or movement that may cause injury, do not use if unsure.
- If one or more of the bed functions is found not working correctly, notify the manufacturer for advice.

Every 6 Months:

- Check all accessories including rails are still functioning properly.

Annually:

- Professional service of product to be carried out annually & documented in 'Record of Performed Service Procedures and Regular Service Maintenance in the back of this instruction manual.

