

Staebel Toilet Surround User Manual

Product Code: 190



Congratulations on your purchase of an Staebel quality product.

Before using your product, please read all instructions carefully. The warranty will not apply if the product has been used incorrectly. This product has been tested to a safe working load of 125-150kg. Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

INTRODUCTION

Intended Use

For use in a bathroom or shower, to fit around the toilet and provides assistance in getting on and off the toilet for those with limited mobility.

Specifications

Product Code	1900	1901	1902
Width Between Arms	460 mm	540 mm	620 mm
Arm Height	685 mm	685 mm	685 mm
Overall Width	540 mm	605 mm	670 mm
Overall Depth	490 mm	520 mm	520 mm
SWL	125 kg	130 kg	150 kg

Safety

- The maximum user weight must not be more than 125-150 kg.
- Ensure all four legs are level and in contact with the shower floor.

ASSEMBLY/INSTALLATION

Unpacking

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

Height Adjustment

Adjust to the correct height by removing the pin clips, sliding the leg extensions up/down the frame tube to the appropriate height and replace the pin clip. Ensure all four leg extensions are secured to the same adjustment hole to ensure even height.

CLEANING & MAINTENANCE

Maintenance

Inspect at regular intervals for wear and damage.

Cleaning Instructions

Clean regularly, wiping down with mild detergent /disinfectant solution. Allow product to fully dry prior to use/storage.

PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

WARRANTY

Staebel warrants this product for a period of **12 months**, from the date of purchase, against faulty materials and workmanship.

This warranty does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only. Any guarantee issued by Staebel does not affect your statutory rights.