

# StaebeL P5500 Bed Extension User Manual

Product Code: 7-C.EXT5000



**Thank you for choosing the Staebel P5500 Bed Extension.** Please carefully read these instructions before using the accessory. This instruction manual contains information on the instructions for use on operating this accessory.

These instructions for use may contain inaccuracies or printing errors. The information provided here will be updated periodically and changes due to product maintenance will be included in later editions. Changes or improvements are possible at any time without prior notice. If you have any questions, please contact our customer service team.

The instructions for use must be read and used by every person who operates this accessory.

## PURPOSE

The intended purpose is to be used for individuals requiring extra length. Users must be instructed on its correct use and familiarise themselves with these instructions.

## PRE-OPERATION CHECK

### Please check the following:

- Make sure this product doesn't interfere with moving bed parts.
- When installing the bed extension to this bed, please beware of the possible crushing points.

### Important:

Warning: The word Warning refers to a hazard to unsafe practice that may cause severe injury or death to you or another person. Accident risk Warning: Do not use or allow use where the intended user has;

- Cognitive impairment
- History of recurrent falls
- Uses medication that would compromise falls
- Limited mobility comprises safe use

Designed to be fitted to P5500 Homecare Bed. Failure to comply may result in warranty being void.

## MAINTENANCE

- Accessory must be inspected regularly for any wear or movement that may cause injury, do not use if unsure.

### Every 6 Months:

- Check all accessories including are still functioning properly.

### Annually:

- Professional service of product to be carried out annually & documented in 'Record of Performed Service Procedures and Regular Service Maintenance in the back of this instruction manual.

## RECORD OF PERFORMED SERVICE PROCEDURES AND REGULAR SERVICE MAINTENANCE

| Date | Purpose and Description | Performance |
|------|-------------------------|-------------|
|      |                         |             |
|      |                         |             |
|      |                         |             |

## FITTING THE P5500 BED EXTENSION



**1.** Remove foot end Hi/Low and knee break cables from the control box (positions below)

|                |                         |
|----------------|-------------------------|
| <b>Port 1</b>  | Backrest Actuator       |
| <b>Port 2</b>  | Foot End Hi-Low Section |
| <b>Port 3</b>  | Kneebreak Actuator      |
| <b>Port 4</b>  | Head End Hi-Low Section |
| <b>Port HB</b> | Handset                 |



**2.** Loosen the 4 black knobs Located under the sleeping surface



**3.** Split Bed Apart



**4.** Insert extension, tighten all 4 black knobs and re-connect cables. Test Bed Extension before use.

## WARRANTY

### Conditions

Your Staebel P5500 Bed Extension is guaranteed for **1 years** from the date of purchase. Professional service of product to be carried out annually. This warranty applies only on the parts manufactured or approved by Staebel.

### Exclusive

This warranty does not extent those items that may need replacement due to normal wear and tear, or to damaged caused by misuse or accident, for which Staebel or it's distributors cannot be held responsible.

If your accessory should need attention due to failure as defined under the warranty above, please contact the distributor from whom you purchased your accessory.