

Staebel Disposable Rolling Slide User Manual

Product Code: 210134



Thank you for choosing the Staebel Disposable Rolling Slide. All your repositioning needs with ease: up/down/turning/swivelling/transfers. Please read this instruction booklet carefully before use to avoid any potential accidents and injuries due to misuse.

USER INSTRUCTIONS



1. Make sure that the brakes are on the bed/trolley before commencing a movement. Insert your Tubular Slide Sheet by either (a) 'Log-rolling' the patient/ client to one side, (b) Unrolling directly beneath the patient (top to bottom), or (c) the method you have been trained in.

Please remember: a slide sheet that covers the whole body for dependent patients makes handling easy



2. **To move a patient up the bed:** Carers face the bottom of the bed and grasp the top layer at the hip and shoulder areas (as shown). Then the carers slide the top surface over the bottom surface in the direction of the headboard and the patient will slide up the bed.

NB: Carers - keep your hands gliding on the mattress, do not lift.



3. **To move a patient down the bed:** Carers face the head of the bed and grasp the top layer at the hip and shoulder areas (as shown). Then the carers slide the top surface over the bottom surface in the direction of the feet and the patient will slide down the bed.

NB: Carers - keep your hands gliding on the mattress, do not lift.



- 4a. **To turn a patient:** Carers face each other, agree which way they are turning the patient and then position the patient for turning. The carer to whom the patient will be turned, helps to keep the patient in the centre of the bed, whilst the carer who is turning the patient away from them, gathers the top layer of the Tubular Slide Sheet close to the hips and shoulders, and gently pulls the slide sheet towards them, keeping the slide sheet level, not pulling up.



- 4b. **To turn a patient:** The patient will then turn on their own axis and be positioned safely and centrally in the bed.

NB: Carers - keep your hands gliding on the mattress, do not lift.



5. **Removing the Slide Sheet:** Once the patient has been repositioned the slide sheets must always be removed unless clinical advice for the person in question overrides this.

To Remove: Insert your hand between both layers of the roller slide sheet and invert by withdrawing the furthest bottom edge through the middle of the sheet. This effectively 'peels' the sheet out without creating any shearing effects.

MATERIAL CONTENT

100% Polyester

STORAGE AND HANDLING

- Store in a clean dry environment that is not subject to excessive heat.
- Keep away from sunlight.
- Store flat and avoid placing heavy objects on top of the product

WARNINGS AND PRECAUTIONS

- Do not leave or drop on the floor; it is a slip hazard.
- Inspect product before use for signs of damage or tears.

CLEANING INSTRUCTIONS

- Wipeclean, if needed for the same patient, with natural water and detergent or wipes that don't contain alcohol, solvents, bleaching or abrasive agents.
- Cleaning materials used should be patient safe and biodegradable.

DISPOSAL INSTRUCTIONS

- Non-recyclable.
- Dispose of as clinical waste if it has been used by an infectious person.
- Otherwise dispose of through normal waste management.

