

Staebel Easy Dress Arms User Manual

Product Code: 2101EDA



Thank you for choosing the Staebel Easy Dress Arms. Our Staebel Easy Dress Arms makes getting dressed much easier, both for independent users and those who need support from a carer. Please read this instruction booklet carefully before use to avoid any potential accidents and injuries due to misuse.

USER INSTRUCTIONS


This is an illustrated method for enabling one or two carers to dress someone in bed using the Staebel Easy Dress Arms. If only one carer is available, the process just takes a little more time depending upon the capability of the person being dressed and if they can assist in anyway at all. *PLEASE REMEMBER: If the person being dressed can assist or participate in anyway, please encourage this as it can aid re-enablement as well as encourage their sense of well being and involvement.*



1. Slide the arm sleeve gently over the patient's arm and up towards the shoulder, sliding the extra fabric under the patients back. Use a flat upwards palm and the slide sheet underneath to assist you. Next, slide the clothing sleeve over the arm sleeve, as far up as you can, tucking the back part gently under the person. Again the Flat Slide Sheet will help here. The person may have to turn slightly.



2. Once you have the garment in place, remove the arm sleeve and repeat with the second arm and then remove. At this stage it can sometime be useful to roll the person gently towards you to straighten out the clothing, or as shown, use a small Slippy Sleeve to help iron out the creases. This is dependent upon the person's condition.

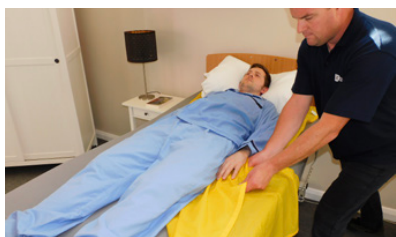

3. Finally, remove the Flat Slide Sheet as well as the arm sleeves.

MATERIAL CONTENT

- 100% Polyester

STORAGE AND HANDLING

- Store in a clean dry environment that is not subject to excessive heat.
- Keep away from sunlight.
- Store flat and avoid placing heavy objects on top of the product.

WARNINGS AND PRECAUTIONS

- Do not leave or drop on the floor; it is a slip hazard.
- Test for slippiness before use. If over-laundered and doesn't slide please discard or replace.
- Inspect product before use for signs of damage or tears.

CLEANING INSTRUCTIONS


- Wash as per laundry instructions.
- Can be wiped clean, between use with the same patient, with natural water and detergent or wipes that don't contain alcohol, solvents, bleaching or abrasive agents.
- Cleaning materials used should be patient safe and biodegradable.

DISPOSAL INSTRUCTIONS

- Non-recyclable.
- Dispose of as clinical waste if it has been used by an infectious person.
- Otherwise dispose of through normal waste management.



Keep Dry

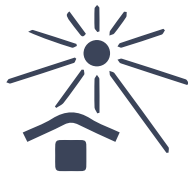


Keep away from direct
sunlight





Caution



Do not use if
package is damaged

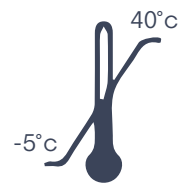

Storage Temperature
Limit

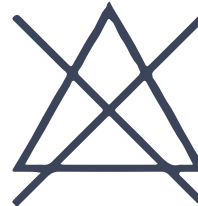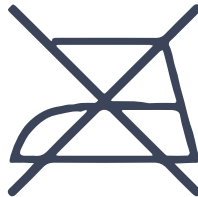

Wash at 30-74°C. Tumble dry on low heat. Do not Iron. Do not use bleach. Do not use fabric conditioner.