

# Staebel In Bed Sit Support User Manual

Product Code: 210440




**Thank you for choosing the Staebel In Bed Sit Support.** Encouraging patients to sit themselves up in bed/lower themselves into bed in a controlled way. Please read this instruction booklet carefully before use to avoid any potential accidents and injuries due to misuse.

## USER INSTRUCTIONS



- 1. Hospital/Community Profiling Beds:** Thread the long loop down through the bed frame and back.

**Divan Beds:** Thread the large loop round the bed leg/castors.

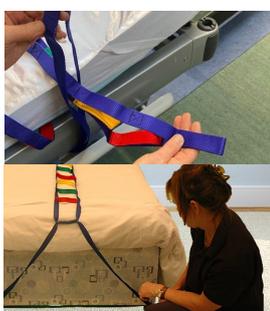

- 2. Hospital/Community Profiling Beds:** Thread the rest of the ladder through the loop and pull tight. The ladder is then securely held.

**Divan Beds:** Check that they are secure and position the Webbing Rope Ladder close to where the person needs it.



- 3.** Using the different rungs of the ladder the user can help themselves up or down. The multi coloured rungs are comfortable and easy to grip.



- 4.** The Webbing Rope Ladders can give extra security to people getting in and out of bed.

## MATERIAL CONTENT

- 100% Polypropylene

## STORAGE AND HANDLING

- Store in a clean dry environment that is not subject to excessive heat
- Keep away from sunlight
- Store flat and avoid placing heavy objects on top of the product

### WARNINGS AND PRECAUTIONS

- Do not leave or drop on the floor; it is a slip hazard
- Inspect product before use for signs of damage or tears
- • Safe Working Load (SWL): 300kg

### CLEANING INSTRUCTIONS


- Wash as per laundry instructions
- Can be wiped clean, between use with the same patient, with natural water and detergent or wipes that don't contain alcohol, solvents, bleaching or abrasive agents
- Cleaning materials used should be patient safe and biodegradable

### DISPOSAL INSTRUCTIONS

- Non-recyclable
- Dispose of as clinical waste if it has been used by an infectious person
- Otherwise dispose of through normal waste management



Keep Dry

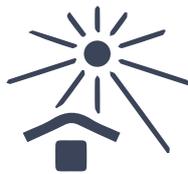


Keep away from direct sunlight





Caution



Do not use if package is damaged

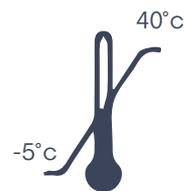

Storage Temperature Limit

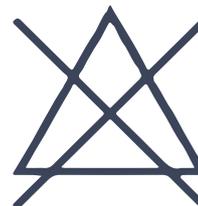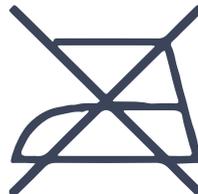

Wash at 30-74°C. Tumble dry on low heat. Do not Iron. Do not use bleach. Do not use fabric conditioner.