

Staebel Deluxe Slidesheet User Manual

Product Code: 2434



Thank you for choosing the Staebel Deluxe Slidesheet. The ideal solution when transferring or repositioning aged care residents across, up or down the bed.

Do not hesitate to contact us in case of queries about the operation or maintenance of your Staebel equipment. Thoroughly read and understand this user manual before using your slidesheet. Information from this user manual is essential for proper operation and maintenance.

CLEANING INSTRUCTIONS

Made from a strong, non flammable silicone coated polyester taffeta these premium quality sheets retain their slip for over 150 machine washes. The Deluxe Slidesheet will retain their colour, lustre, softness, and functional integrity if laundered as follows:

- Separate Deluxe Slidesheets from other Linen
- **Program:** Select gentle wash cycle
- **Duration:** 5 to 10 minutes (do not over wash)
- **Chemical:** Woolwash detergent pH 7
- **Water Temp:** 65°C - 75°C
- **Extract:** Slow- Medium
- **Rinse:** 65°C - 75°C
- **Spin:** Slow- Medium
- **Dry:** Warm tumble (not hot) or air dry, short duration

*Wool wash delicate recommended

*Strictly no eucalyptus or tea tree additives to be used

*Avoid the use of alkaline detergents, bleaches and scouring agents.

SPECIFICATIONS

Specifications	
Product Code	2434
Width	1450 mm
Length	2000 mm

Material: Non-flammable silicone coated polyester taffeta