

Staebel Patient Transport Chair User Manual

Product Code: 4750



Information before using the product / the functioning of medical products

1. Before use carefully read the enclosed instructions for use.
2. Use the product only for its intended purpose.
3. Product can be used only by personnel who are familiar with the functioning of the product.
4. The maintenance, as well as the safety and technical check-ups, can be carried out only by an authorized service company or by any other professional company authorised by Staebel.
5. Assembly, revisions or repairs can be carried out only by an authorised service company or by any other professional company authorised by Staebel.
6. If technical modifications, including upgrades or revisions of the products, are not carried out by Staebel or any other professional service company authorised by Staebel, any guarantee for such revisions, as well as for products on which such revisions were carried out, shall be deemed void.

Preventive measures

Please read the instructions for use carefully. Use the product only for the intended purpose. Always follow the notes under the »WARNING« signs.

Intended use

A wheeled personal mobility chair is intended for transporting patients over a short distance. It has small wheels that the patient cannot use for propulsion. It is designed for a person with a disability or a person without the full capacity to walk. Chair also includes foot plate to keep the patient's feet raised from the ground. The device is intended for use in environment of care, hospitals, nursing homes, rehabilitation units and other healthcare settings. It can not be folded or readily dismantled for transport.

Regulations and norms

Always ensure compliance with valid norms or regulations.

CE mark

CE mark refers to the Directive 93/42/EEC for medicinal products and MDR 2017/745.

Responsibility and guarantee

The manufacturer of the Patient Transport Chair is responsible for the safety and reliability of the product provided that:

- The product is used solely for its intended use.
- The functional check-ups are carried out on a regular basis.

Transport damages

Upon delivery, please, check the product and its packaging for damages. Always refer to the delivery date: visible damages should be reported within 2 days after delivery and hidden damages within 7 days after delivery.

Serial number

The serial number is located on the product label, which is on the inner side of the product left leg.

Product display – Staebel Patient Transport Chair

- | | |
|---------------------------|-------------------------------|
| 1. Back rest upholstery | 7. Handbrake |
| 2. Seat upholstery | 8. Double-wheel caster Ø100mm |
| 3. Movable armrests | 9. Caster with brake Ø300mm |
| 4. Belongings box | 10. IV pole bracket |
| 5. Retractable foot plate | 11. PVC pocket for documents |
| 6. Maneuvering handle | 12. Seatbelt bracket |



Technical Data

Product Code	4750
Width	660 mm
Length	870 mm
Height	1090 mm
Seat Height	620 mm
Seat Width	520 mm
Seat Depth	430 mm
Weight	28.5 kg
SWL	200 kg

First use

The Patient transfer chair is ready for use immediately after removing the packaging.

Product use:

Engage the park brake

Engage the rear wheel brake by squeezing on the hand brake handle until the red locking mechanism lock into place and brake shoe presses firmly to prevent the chair from rolling.



Release the park brake

Press on the red locking mechanism to unlock it and slowly release brake handle to release the rear wheel brake.

Movable armrests

Open up the two armrests by manually pulling them upwards.

Retractable foot plate

Retractable foot plate is adjustable by sliding in and out.

Step by step use

- Always inform the patient about what you are planning to do.
- Engage the rear wheel locks by pushing down on the brake handle until the brake shoe presses firmly to prevent the chair from rolling.
- Retract the foot plate under the belongings box so that the user can stand with the back of their legs touching the seat upholstery.
- From behind the chair, assist the user by slowly lowering them into the chair.
- Extract foot plate so patient can place the feet on the foot plate.
- Release the brakes and proceed with the transfer.

WARNING

- To prevent patients from falling during accommodating always apply the brakes before using the equipment. (See Fig. 1)
- To avoid injuries, make sure that the patient is never left unsupervised.
- To prevent patient falls or caregiver injuries make sure that there are two caregivers present during the transfer of a patient if this is necessary according to the patient state.
- To prevent cross-contamination disinfect regularly between each use, if necessary, or daily, as a minimum.
- To prevent injuries always close movable armrests after accommodating the patient.



Fig. 1

Maintenance

The product is recommended to be cleaned and/or disinfected regularly between each use, if necessary, or daily, as a minimum. Clean the product with a damp cloth using warm water and a disinfectant cleaner. Disinfectant wipes, which are pre-soaked with a 70% (v/v) solution of isopropyl alcohol, can also be used. Knee and seat supports are made of composition leather, which is easy to maintain and complies with DIN 54020. This type of coating can be wiped with a dry or damp cloth; detergent for sensitive laundry can also be used, if necessary, but it must be wiped immediately after use (the stains should be removed as soon as possible to prevent its absorption by the material). Finally, the material should be wiped with warm and clean water.

Safety and technical control

Pursuant to the "Medical Product Act" the safety and technical checks should be carried out once a year.

Repair, Servicing and Spare Parts

In case of any technical problems, please contact Staebel or your local distributor.